RC-CHAMBER®

Optimiert die Inhalation mit Dosieraerosolen
Optimized inhalation with metered dose inhalers
Optimiza la inhalación con inhaladores de cartucho presurizado
Inalazione ottimizzata con inalatori-dosatori
Inhalation optimisée avec les inhalateurs-doseurs

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THANK YOU FOR CHOOSING THE RC-CHAMBER® THERAPY.

NOTE

Please read the Instructions for Use carefully before using RC-Chamber® for the first time. Please also note the instructions for the metered-dose inhaler. Please keep these instructions for future reference. Please note that the device is designed for single-patient use only! Do not allow children to use with RC-Chamber® without supervision. An adult should always be present when therapy is administered to children and those requiring assistance. All RC-Chamber® parts can be sterilized in boiling water and disinfected in a steam sterilizer as well as cleaned in the RC-Clean cleaning bag. (To prevent the build-up of lime-scale, we recommend boiling the device in distilled water.) Store RC-Chamber® in a dry, dust-free environment and protect it from sunlight. We recommend the exchange of the RC-Chamber® after one year of using. In Germany the RC-Chamber® is reimbursable once a year on submission of a medical prescription. Disposal: RC-Chamber® and its accessories may be discarded with household waste unless otherwise stipulated by the regulations of the respective country of use.

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⚠️ WARNINGS

Please clean the device prior to using it for the first time. Clean the device once per week if used regularly, or more frequently as needed. If the device is visibly soiled, clean RC-Chamber® immediately prior to or after use. Replace RC-Chamber® after one year if used daily. Please ensure that RC-Chamber® is used properly according to the Instructions for Use. Incorrect application (off-label use) results in exclusion of liability for the manufacturer.

CONTENTS

The package contains RC-Chamber®, consisting of:

- Mouthpiece
- Valve & inhalation controls - Cross Valve Technology™
- Spray entrance chamber
- Back piece: clear plastic ring with integrated rubber adapter

e. A mask is also included with the following articles:

- RC-Chamber® with mask for tracheostomized patients
- RC-Chamber® with mask for infants
- RC-Chamber® with mask for young children
- RC-Chamber® with mask for adults and older children

f. Emergency plan (to stick to the back of the security bag)

g. Guardian angel pendant (to attach to the security bag, perforation above the pressure lock necessary)

Materials:
Chamber: Makrolon polycarbonate resin
Back piece: Santoprene
Valve & inhalation controls as well as masks: Silicone
All materials are latex-free.

Safety breathing concept

A reliable companion throughout the day and in an emergency

Included in delivery RC-Chamber® including emergency plan and guardian angel pendant. Drugs as well as the Peak Flow Meter Personal Best® must be ordered separately.

Further information and films on the RC-Chamber® and to the emergency safety concept you receive as Video or for reading on www.cegla.de

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AVAILABLE PRODUCTS

RC-Chamber® with mask for infants up to 1 year
Article no. 30100 | PPN 111 171 099 278

RC-Chamber® with mask for young children, 1-5 year
Article no. 30200 | PPN 111 171 100 087

RC-Chamber® with mouthpiece for adults and older children
Article no. 30300 | PPN 111 171 102 343

RC-Chamber® with mask for adults and older children
Article no. 30400 | PPN 111 171 101 777

RC-Chamber® for tracheostomized patients
Article no. 30450 | PPN 111 273 164 894

MASKS

Masks available separately.

Infant mask
Article no. 30110 | PPN 111 416 498 106

Child mask
Article no. 30210 | PPN 111 416 499 893

Mask for adults and older children
Article no. 30410 | PPN 111 416 500 605

RC-Tracheal mask for tracheotomized patients
Article no. 3012

ACCESSORIES AND ADDITIONAL PRODUCTS

RC-Clean
Reusable microwaveable cleaning bag. Easy and quick cleaning of medical products as well as baby items.* Use only for microwave safe products.
Article no. 5000 | PPN 111 075 103 265

* Report on microbiological testing available on request.

Replacement parts and additional products are available in pharmacies, medical supply stores, or at www.cegla-shop.com

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To constantly improve our level of service, we welcome your feedback. You will find helpful tips for living with respiratory diseases on our platform www.leichter-atmen.de. We look forward to your visit.

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All trademarks and registered trademarks belong to the company R. Cegla GmbH & Co. KG, Germany. In some instances, product graphics may vary from actual product models, sizes and/or colors listed below.

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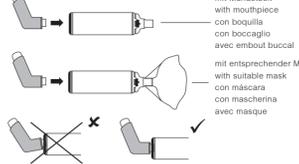
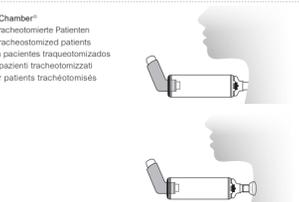
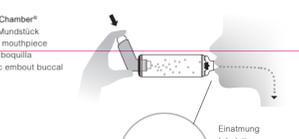
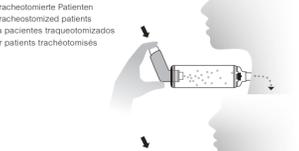
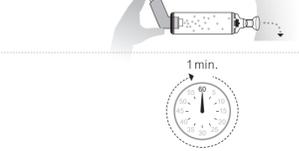
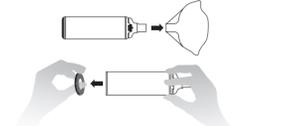
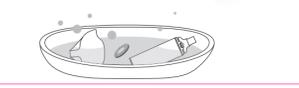
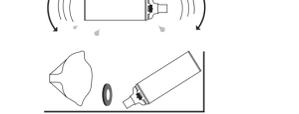
CEGLA
MEDIZINTECHNIK

Medical Developments International (MDI)
4 Caribbean Drive,
Scoresby, Victoria 3179,
Australia

Masken | mask | máscara | maschera | masque
Vertrieb | Sales | Distribution | Distribuzione | Distribué par:
R. Cegla GmbH & Co. KG
Horresser Berg 1
56410 Montabaur
Germany
Tel +49 26 02 92 13-0
info@cegla.de
www.cegla.de

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Eigentümerin der Trade Marks und der registrierten Trade Marks ist das Unternehmen R. Cegla GmbH & Co. KG.